



## **Carousel Sliding Bath Bench with Swivel Seat**

### **Model PHB3300/PHB3400G**

## **ASSEMBLY AND OPERATION MANUAL**

Thank you for investing in Carousel Sliding Bath Bench with Swivel Seat. Safe and comfortable bathing is only a few steps away. Please be sure follow all instructions carefully to ensure safe and trouble-free operation for years to come.

## **IMPORTANT SAFETY INFORMATION**

### **READ ALL SAFETY INFORMATION BEFORE USING**

It is the users responsibility to ensure the appropriateness of this device for the user in question before using.

- NEVER use this device with individuals who are severely mentally, cognitively, physically or vision compromised.
- ALWAYS attach safety belt and adjust to a snug fit around users hips.
- NEVER leave a compromised user unattended.
- ALWAYS ensure that the legs are properly adjusted to ensure that the bench is level side to side and front to back.
- NEVER exceed the maximum weight capacity of 330 lbs.

**NOTE:** Failure to follow the above noted safety instructions can lead to severe injuries.

## **ASSEMBLY**

### **STEP 1**

Remove all parts from the box and remove all plastic packaging. Ensure all parts are present as shown here. In the event of missing parts, please call customer service at 1-888-550-2410.



### **STEP 2**

Insert the curved upper leg sections into frame of shower chair as shown. Slide parts together until the brass tension spring buttons pop out of the holes.

**NOTE:** only two tension spring buttons will show the other two tension spring buttons need to be pressed so the legs slide into the tube.




### STEP 3

Attach one lower leg section to each corner of the frame as shown.



### STEP 4

Insert the backrest into the frame of the sliding chair until the spring tension buttons engage in the hole.

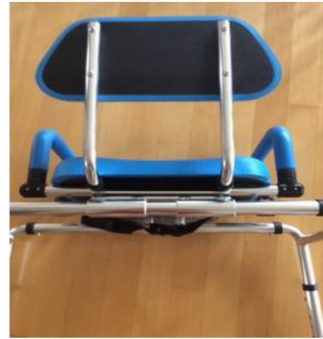

### STEP 5

Adjust legs so they will best accommodate floor heights and desired height for client. Legs lengths are adjustable in half-inch increments by pushing the spring tension buttons. Ensure that seat is level from front to back and also side to side.



## OPERATION

### STEP 1

Push the gray button straight in engage the chair sliding function. Slide the chair along the horizontal rails until it locks into place at the end of the travel outside of the tub.




### STEP 2

To rotate the chair, push down on the red lever. Chair will lock into place every 90° of rotation. Rotate the chair so that it is facing into the room and toward the user. Have user sit onto chair and securely fasten safety belt around hips.



Ensure that the safety belt is securely attached and snug around the hips. NOTE: Never use the carousel chair without attaching the safety belt. Using chair without safety belt can lead to serious injury. NOTE that for easier transfers, the armrests of the Carousel can be raised. To raise armrests, press red button at bottom front of armrest. This will unlock the armrest so that it can be moved out of the way. Once seated, lower armrest back into position until it locks into place.

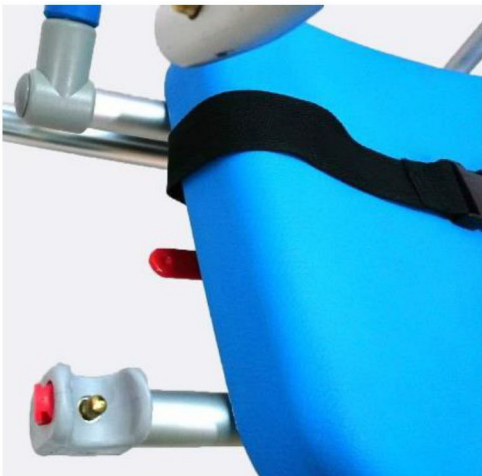

### STEP 3

Press the red lever and rotate the chair while user lifts one leg into the tub at a time. Once chair is fully rotated and locked into position with the user facing the front of the tub, press the gray button under the seat. This will disengage the chair so it can slide along the horizontal rails into the tub. Slide the chair to the end of travel until it locks into place.



### STEP 4

Bathe normally.



## STEP 5

To exit the tub, press the gray button to disengage sliding chair and slide the chair toward the outside of the tub to the end of the travel. Once the chair locks in place, press the red lever and rotate the chair lifting one leg at a time out of the tub. Ensure users feet are on a dry surface then unlock the safety belt so user can stand and exit the chair.

**NOTE** that the Carousel Sliding Transfer chair can be used in a tubs and showers.

**NOTE** that the chair can rotate 360° and can therefore be used with right hand openings and left hand openings, it is completely reversible.

## TROUBLESHOOTING

### PROBLEM

Chair not sliding easily or smoothly.

### SOLUTION

Note that a light lubricant is applied to the horizontal rails that the chair slides on. This lubricant is applied at the factory before the item is shipped out. In some cases this lubricant can evaporate if high temperatures are encountered during the shipping process. To rectify the problem, simply apply a light coating of WD 40 lubricant to the horizontal rails.

### PROBLEM

Chair is not completely level in the shower or bathtub.

### SOLUTION

Rotate the rubber feet to accommodate the contour or shape of the tub or shower, making sure they are flat and in full contact to the surface of the tub or shower. Rubber feet are angled so that they can be adjusted to compensate for irregular or un-even surfaces. Also, ensure that the brass tension knobs on the leg adjustments are positioned at the correct level so that the chair is completely level.

For additional information or assistance, please contact us anytime, toll free at 888-550-2410. We can also be reached by email at [info@platinumhealthllc.com](mailto:info@platinumhealthllc.com).

Copyright 2019. Platinum Health LLC

